

**1**

## Correct size and fit

- Apply product as shown.
- Leg enclosures should rest at the upper most thigh area, avoiding any gaps.

**4**

## Cut back on barrier creams

- If barrier cream is necessary, apply a thin layer and rub in like lotion on hands.



**2**


## Check regularly

- Check per facility protocol
- Check for bowel movement
- Check wetness indicator on brief

**3**

## Change only when needed

- Change only when indicator lines are blue within 1" to 2" from either end.
- Always change if resident has had a bowel movement.

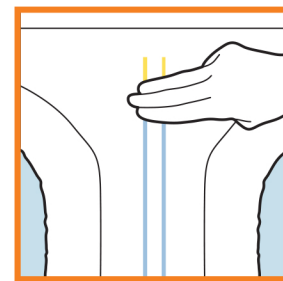

View the video "4C's of Incontinence Care" at [www.TranquilityClinical.com](http://www.TranquilityClinical.com)

**TRANQUILITY**  
PREMIUM PROTECTION

**SELECT**  
Incontinence Products

**ComfortCare**  
Incontinence Products

For additional information or questions  
Call: 1-800-467-3224 x 340 or email: [clinical@pbenet.com](mailto:clinical@pbenet.com)