

## The Tranquility Difference

The **Tranquility Product Family** of incontinence products are designed for **When Performance Matters Most**. All products are carefully engineered with a unique blend of superabsorbent polymers to lock away fluid and odor, keeping wearers' skin dry. Products are organized by absorbent capacity ranging from maximum protection of Tranquility, to the heavy protection of Select, to the moderate protection of ComfortCare.

## New Patient Transition


Incontinence product transitions commonly create sizing challenges as there is not an industry standard for sizing/fit. Specifically, Tranquility products offer individual unisex sizes that have a more generous fit. If transitioning a patient from a combined size product (i.e. S/M), there could be a variance in selecting the correct size. Tranquility recommends following the below guidelines to properly measure and fit new patients.

## Sizing Guide

Suggested sizes are a guideline. When in doubt, best practice is to use a tape measure and obtain an accurate hip measurement. **For sizes greater than an XL, obtain an accurate hip measurement for best fit.**

### How to Measure

Use a tape measure to measure in inches the widest part of your body between waist and hip. Use this number to determine product size.

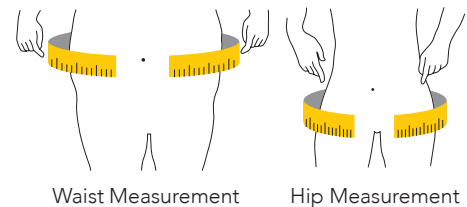

## Disposable Absorbent Underwear

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	15" - 25"	38 - 65 lbs	N/A	N/A
X-Small	17" - 28"	65 - 85 lbs	0-3	17-28
Small	22" - 36"	80 - 125 lbs	4-6	22-36
Medium	34" - 48"	120 - 175 lbs	6-12	34-48
Large	44" - 54"	170 - 210 lbs	12-18	44-54
X-Large	48" - 66"	210 - 250 lbs	18-24	48-66
2XL	62" - 80"	250+ lbs	24+	62+

### Correct Fitting DAU



#### Correct Fit

Product should fit snugly, like underwear, touching the crotch with no gaps around the legs.

### Poor Fitting DAU



#### Too big

Gaps around legs, underwear not touching the crotch will cause leakage.



#### Too small

Too tight around waist and legs, may cause irritation.



For more information, samples, or to trial a product at your facility, please call customer relations 1-800-467-3224 x7.