

New Name, New Look, Same Great Features!

INTRODUCING

Tranquility® Essential Booster Pads



Coming October - December 2021!



Previously Known As



Is Now



essential

Select® and ComfortCare™ Booster Pads will start to transition in October - December 2021 with a new name and refreshed packaging.

The names of these products will transition to our new Tranquility Essential Booster Pads. Available in Heavy and Moderate Protection.



FEATURING

Flow-Through Design • Clothlike Adhesive Strip • Latex-Free



What is a Booster Pad?

Booster pads have a flow-through design allowing them to fill to capacity before passing additional moisture into a primary incontinence product. Booster pads should always be placed inside another disposable incontinence product that has a moisture-proof backing. **Do not use with regular underwear.**

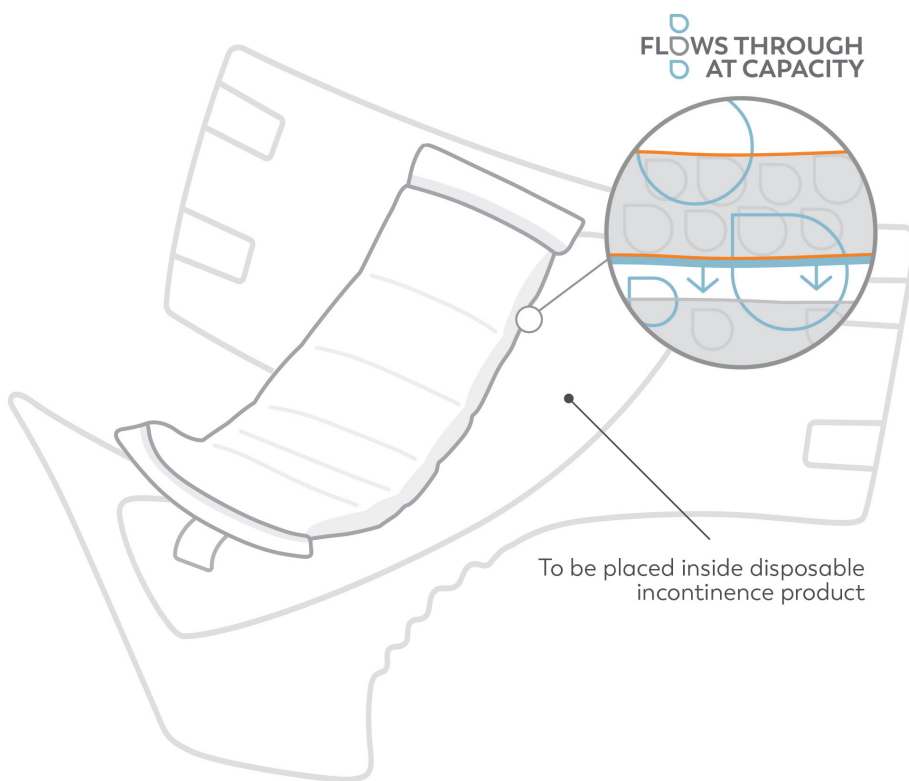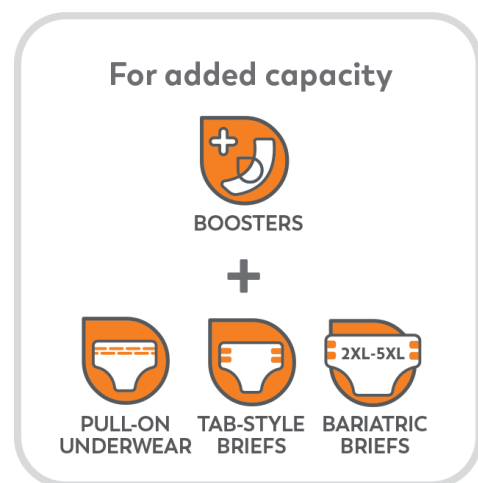


Did You Know?

Booster pad placement is important. **The pads can be placed in targeted areas where additional absorption is needed.**



Booster pads can be folded and even doubled for added protection, but should never be cut.

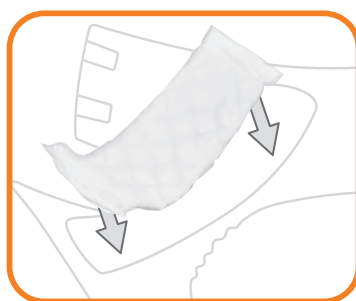

Host Garment - The pad's adhesive side should face out, away from the body. Make sure the pad is fully contained within the host incontinence product; no part should extend beyond the edges.

For Women - Place the pad more toward the bottom.

For Men - Place the pad more toward the front.

