

# New Name, New Look, Same Great Features!

## Tranquility® Essential Booster Pads

.....



Previously Known As



Is Now

**essential**

Select® and ComfortCare™ Booster Pads have transitioned with a new name and refreshed packaging.

The names of these products have transitioned our new Tranquility Essential Booster Pads. They are available in Heavy and Moderate Protection.



### FEATURING


Flow-Through Design • Clothlike Adhesive Strip • Latex-Free



# What is a Booster Pad?

Booster pads have a **flow-through design** allowing them to absorb initial fluid before passing additional moisture into a primary incontinence product. Booster pads should always be placed inside another disposable incontinence product that has a moisture-proof backing. **Do not use with regular underwear.**

FLOW-THROUGH DESIGN

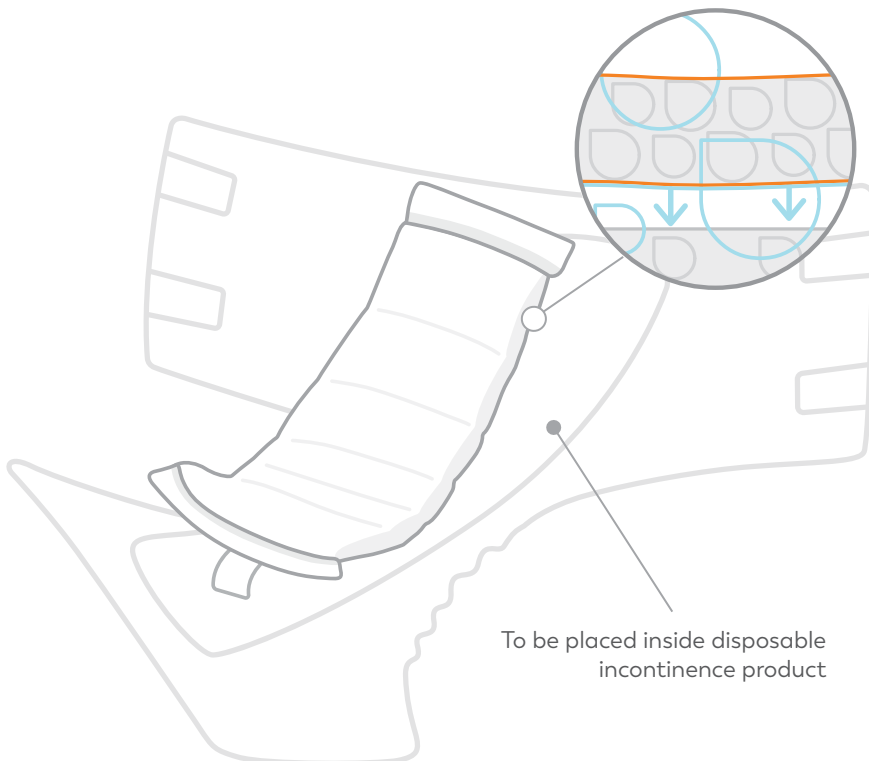

For added capacity



BOOSTERS

+



PULL-ON UNDERWEAR



TAB-STYLE BRIEFS




BARIATRIC BRIEFS

## Did You Know?

Booster pad placement is important. **The pads can be placed in targeted areas where additional absorption is needed.**



Booster pads can be folded and even doubled for added protection, but should never be cut.

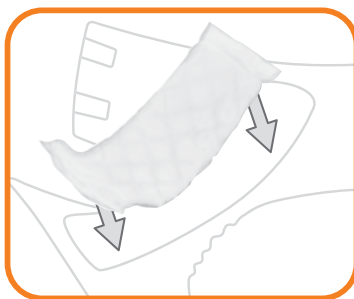

**Host Garment** - The pad's adhesive side should face out, away from the body. Make sure the pad is fully contained within the host incontinence product; no part should extend beyond the edges.

**For Women** - Place the pad more toward the bottom.

**For Men** - Place the pad more toward the front.

