# **Sizing Guide**

# WIDEST

#### How to Measure

Use a tape measure to measure in inches the **widest part of the body between waist and hip.** Use this number to determine product size. To measure an individual who is **lying down**, see instructions on reverse.

## 了 Briefs Size Guide

Size	Waist/Hip	Weight	Women's Pant Size	Men's Pant Size
Youth	16" - 22"	28 - 42 lbs	N/A	N/A
X-Small	18" - 26"	42 - 90 lbs	0 - 3	18 - 26
Small	24" - 32"	85 - 100 lbs	4 - 6	24 - 32
Medium	32" - 44"	100 - 150 lbs	8 - 12	32 - 44
Large	44" - 58"	150 - 200 lbs	14 - 18	44 - 58
X-Large	56" - 64"	200 - 250 lbs	18 - 24	56 - 64
2XL	60" - 80"	250+ lbs	26 - 28	60+
3XL	64" - 96"	250+ lbs	28+	64+
4-5XL	70" - 108"	250+ Ibs	28+	70+



### Protective Underwear Size Guide

Size	Waist/Hip	Weight	Women's Pant Size	Men's Pant Size
Youth	15" - 25"	38 - 65 lbs	N/A	N/A
X-Small	17" - 28"	65 - 85 lbs	0 - 3	17 - 28
Small	22" - 36"	80 - 125 lbs	4 - 6	22 - 36
Medium	34" - 48"	120 - 175 lbs	6 - 12	34 - 48
Large	44" - 54"	170 - 210 lbs	12 - 18	44 - 54
X-Large	48" - 66"	210 - 250 lbs	18 - 24	48 - 66
2XL	62" - 80"	250+ lbs	24+	62+
3XL	75" - 95"	250+ lbs	28+	64+



Sizing is not always perfect; the sizes shown are recommendations, not a guarantee.




For questions, call Tranquility Customer Care at 1-866-865-6101 or for on-demand clinical support, visit TranquilityClinical.com.



## Measuring Instructions for Individual Lying Down



- While the individual is lying on the bed (supine position), **measure at the widest part of the hips** sheet to sheet.
- Start the tape measure touching the bed (A). Run the tape measure over the individual's widest part to the opposite side. End touching the bed, directly next to the individual's body (B).
- This number represents **half** of the individual's waist size.

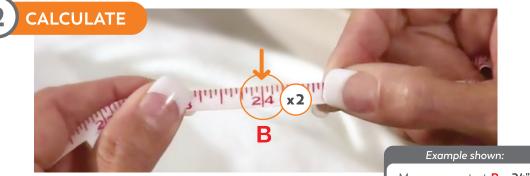

 Double that measurement (Bx2) to determine the individual's total waist size. Measurement at **B** = **24**" 24" x 2 = **48" total waist** 

• Record this total waist measurement on the individual's chart.

#### FIND SIZE

- Compare the final number with the sizing guide.
- If the number falls between two sizes, start with the smaller size.

48" Medium _32" - 4		Example shown:
		<b>48"</b> = Size <b>Large</b> Brief
(	Large	44" - 58"
	X-Large	56" - 64"
	2XL	60" - 80"



For questions, call Tranquility Customer Care at 1-866-865-6101 or for on-demand clinical support, visit TranquilityClinical.com.

