

Usage Guidelines

Tranquility® Premium Underwear (Pull-on)



Purpose	Tranquility products are manufactured with special superabsorbent polymers. Because of their unique design, moisture is pulled into the core of the product keeping the surface against the skin dry and promoting longer wear times.																																																
What to expect	Over time expect to observe: <ul style="list-style-type: none">• Improved skin integrity• Improved sleep patterns for residents (due to longer wear time)• Improved behavioral patterns (Uninterrupted sleep reduces anxiety, irritation and agitation)• Improved environment with the reduction of urine odor• Decrease in changing clothing and linens due to leakage.																																																
When to use	<ul style="list-style-type: none">• For incontinent management (urine and liquid stool)• Most widely used on ambulatory residents.																																																
Sizes and Capacity	<table border="1"><thead><tr><th>Item #</th><th>Description</th><th>Waist/Hip</th><th>Fluid Capacity</th></tr></thead><tbody><tr><td>2113</td><td>X-Small OverNight Underwear</td><td>17" - 28"</td><td>600 mL</td></tr><tr><td>2114</td><td>Small OverNight Underwear</td><td>22" - 36"</td><td>600 mL</td></tr><tr><td>2115</td><td>Medium OverNight Underwear</td><td>34" - 48"</td><td>1005 mL</td></tr><tr><td>2116</td><td>Large OverNight Underwear</td><td>44" - 54"</td><td>1005 mL</td></tr><tr><td>2117</td><td>X-Large OverNight Underwear</td><td>48" - 66"</td><td>1005 mL</td></tr><tr><td>2118</td><td>2XL OverNight Underwear</td><td>62" - 80"</td><td>1005 mL</td></tr><tr><td>2119</td><td>3XL OverNight Underwear</td><td>75" - 95"</td><td>1005 mL</td></tr><tr><td>2105</td><td>Medium DayTime Underwear</td><td>34" - 48"</td><td>740 mL</td></tr><tr><td>2106</td><td>Large DayTime Underwear</td><td>44" - 54"</td><td>740 mL</td></tr><tr><td>2107</td><td>X-Large DayTime Underwear</td><td>48" - 66"</td><td>740 mL</td></tr><tr><td>2108</td><td>2XL DayTime Underwear</td><td>62" - 80"</td><td>740 mL</td></tr></tbody></table> <p><i>Note: It is important to use the appropriate size. If product is too big, leakage may occur. If product is too small, skin irritation may occur.</i></p>	Item #	Description	Waist/Hip	Fluid Capacity	2113	X-Small OverNight Underwear	17" - 28"	600 mL	2114	Small OverNight Underwear	22" - 36"	600 mL	2115	Medium OverNight Underwear	34" - 48"	1005 mL	2116	Large OverNight Underwear	44" - 54"	1005 mL	2117	X-Large OverNight Underwear	48" - 66"	1005 mL	2118	2XL OverNight Underwear	62" - 80"	1005 mL	2119	3XL OverNight Underwear	75" - 95"	1005 mL	2105	Medium DayTime Underwear	34" - 48"	740 mL	2106	Large DayTime Underwear	44" - 54"	740 mL	2107	X-Large DayTime Underwear	48" - 66"	740 mL	2108	2XL DayTime Underwear	62" - 80"	740 mL
Item #	Description	Waist/Hip	Fluid Capacity																																														
2113	X-Small OverNight Underwear	17" - 28"	600 mL																																														
2114	Small OverNight Underwear	22" - 36"	600 mL																																														
2115	Medium OverNight Underwear	34" - 48"	1005 mL																																														
2116	Large OverNight Underwear	44" - 54"	1005 mL																																														
2117	X-Large OverNight Underwear	48" - 66"	1005 mL																																														
2118	2XL OverNight Underwear	62" - 80"	1005 mL																																														
2119	3XL OverNight Underwear	75" - 95"	1005 mL																																														
2105	Medium DayTime Underwear	34" - 48"	740 mL																																														
2106	Large DayTime Underwear	44" - 54"	740 mL																																														
2107	X-Large DayTime Underwear	48" - 66"	740 mL																																														
2108	2XL DayTime Underwear	62" - 80"	740 mL																																														
When to change/replace	<ul style="list-style-type: none">• There is no set time between changes. This depends on variables such as amount of diuretics and/or fluid intake.																																																
Barrier cream	<ul style="list-style-type: none">• Barrier creams are not necessary due to the superabsorbent polymers that draw the fluid away from the skin surface. However, if necessary use a "pea" size amount of barrier cream over the area of concern.• Overuse of barrier cream will block the pores of the product and the fluid will not be able to penetrate though the top layer. This practice may cause leakage.																																																



For questions, call Tranquility at 1-800-467-3224 ext. 363 or for on-demand clinical support, visit TranquilityClinical.com.

