




Usage Guidelines

Tranquility® ThinLiner Moisture Management® Absorbent Sheets



<h3>Purpose</h3>	<p>Tranquility ThinLiners are manufactured using superabsorbent polymers. ThinLiners are composed of ultra-thin material that wicks moisture away from the skin thus providing a dry environment in areas where moisture may cause significant skin breakdown.</p>												
<h3>What to expect</h3>	<p>Over time expect to observe:</p> <ul style="list-style-type: none"> • Improved skin integrity • Decreased odor • Improved comfort 												
<h3>Sizes and Capacity</h3>	<table border="1"> <thead> <tr> <th>Item #</th> <th>Description</th> <th>Fluid Capacity</th> </tr> </thead> <tbody> <tr> <td>3190</td> <td>6" x 10"</td> <td>177 mL</td> </tr> <tr> <td>3191</td> <td>6" x 14"</td> <td>251 mL</td> </tr> <tr> <td>3092</td> <td>20" x 22"</td> <td>450 mL</td> </tr> </tbody> </table>	Item #	Description	Fluid Capacity	3190	6" x 10"	177 mL	3191	6" x 14"	251 mL	3092	20" x 22"	450 mL
Item #	Description	Fluid Capacity											
3190	6" x 10"	177 mL											
3191	6" x 14"	251 mL											
3092	20" x 22"	450 mL											
<h3>Clinical applications</h3>   	<ul style="list-style-type: none"> • Between skin folds, especially under the pannus (abdominal apron). • Between fingers and toes (for contractures of the digits). Fold the small size lengthwise into 4 equal parts and weave between digits. • In the palm of a hand contracture. Cover a rolled up washcloth with the small sized ThinLiner. • Lymphedema (over dressings for extra absorption) • Gluteal folds (Fold the 6x10 size in half the long way and place in the fold to absorb liquid stool and keep surrounding skin dry). • Under breasts • Lining over splints <p>★ Anywhere there is a moisture problem or the area is prone to moisture related infection.</p>												
<h3>When to change/replace</h3>	<p>There is no set time between changes. This depends on variables such as amount of moisture output.</p>												
<h3>Important facts</h3>	<ul style="list-style-type: none"> • Both sides are absorbent. They may be folded if necessary. • DO NOT cut the ThinLiners as this will result in the leakage of the superabsorbent polymers causing the product to fail. 												



For questions, call Tranquility at 1-800-467-3224 ext. 363 or for on-demand clinical support, visit TranquilityClinical.com.

